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CE 0546

DISPOSITIF MÉDICAL

Hyaneb® - Inhalation solution for the removal of mucous secretions

Intended use

The nebulisation of Hyaneb® facilitates the mobilisation of the viscous secretions in the airways due to an osmotic mechanism. The high percentage of salt attracts water and facilitates the hydration of the mucous. Hyaneb® is particularly indicated in patients with cystic fibrosis and in patients with bronchiectasis

Composition

Hyaneb® hypertonic solution for nebulisation contains 70 mg/ml sodium chloride, 1 mg/ml sodium hyaluronate and distilled water.

Contra-indications

Hyaneb® is contraindicated in patients with known hypersensitivity to its components.

Instructions for use

Any jet nebulizer may be used. For optimal results the PARI jet nebulizer is recommended. Vibrating membrane nebulizers can also be used. If using the eFlow® rapid nebulizer system, for an optimal nebulization it is advisable to wet the aerosol membrane prior to each use by simply dipping it in distilled water. Wetting the membrane is not necessary if the device has just been used. Hold the nebulizer handset in the mouth and breathe normally for the duration of aerosol therapy, which is about 10 minutes. Temperature for use: Nebulize the solution at room temperature.

Precautions for use

- The product is for use by inhalation only.
- The first use of the product should take place under the control of a doctor or qualified personnel. The use by children must take place under the supervision of an adult.
- In particularly susceptible subjects, pre-medication with bronchodilators that can help prevent bronchospastic reactions is advisable at the start of the treatment. Pre-medication with bronchodilators must be carried out under the control of a doctor.
- In case of the onset of bronchospasms or persistent cough, interrupt the treatment and inform the treating doctor.
- Do not use the product if the container was damaged or not perfectly closed at the time of first opening.
- Do not use the product after the expiry date indicated on the packaging.

Do not re-use the contents remaining in the vial

Do not mix any other medicine with the hypertonic solution.

Dosage

Use one 5 ml vial twice a day or according to medical prescription. Treatment can be started with a lower volume which can then be increased progressively.

Storage Conditions

Store at room temperature and keep out of reach of children.

Manufacturer
Chiesi Farmaceutici S.p.A.
Via Palermo 26/A
43122 Parma - Italy

Date of last revision: 2018

CE 0546

MEDICAL DEVICE

Caution

Use by

Please follow the instructions for use

STERILE A

Single use

15°C

25°C

Avoid release into the environment

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